

# DELUXE FOAM VISOR INSTRUCTIONS

 stäger


Clear Packaging



1  
The face visor is made of two parts: the PET sheet and the foam strip. Lay your PET sheet on a flat surface, as shown.



2  
Pick up and pinch your foam strip in the middle.

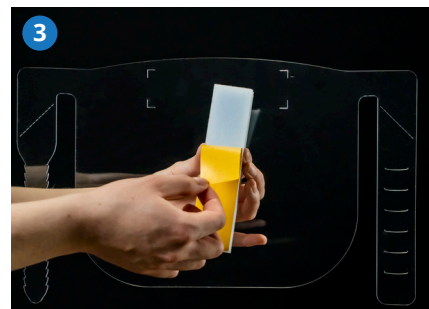


3  
Peel the tab off to reveal the adhesive.



4  
Fasten the foam strip, sticky side down, between the allocated rectangular space on the visor.



5  
Locate the scored line on the visor strap.

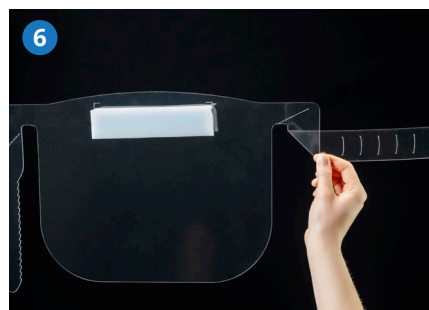

6  
Bend the visor strap under, along this scored line. The strap should be now pointing outwards.



7  
Repeat on the other side.



8  
With both straps folded out, pick-up your face visor and thread one strap through the other (keeping the foam strip on the inside).



9  
Adjust the size by pulling the strip.



10  
Try it on to ensure that it sits comfortably. The foam strip should be on your forehead.

For our technical document head to:  
[staegerclear.co.uk/face-visors](https://staegerclear.co.uk/face-visors)

**Stäger Clear Packaging Limited**  
Unit 1, Swallowgate Business Park,  
Holbrook Lane, Coventry CV6 4BL